



- sensitive feeler arm!

Allowed

- mild soap

The use of the following products / procedures for disinfection will degrade the tip and cause it to fail:



- Alcohol → See page 43 for more information regarding the use of alcohol for disinfection.
- Acetone
- UV radiation
- Sterilization
- Immersing in fluid for more than 1 hour
- Temperature above 60 °C

Maintain the tonometer

- Clean tonometer only with a dampened cloth
- Use soap only

- Do not immerse tonometer in solution!
- No corrosive agents
- No direct spraying



Haag-Streit USA INC
3535 Kings Mills Road
Mason / Ohio 45040

Phone ++ 1 513 336 7255
Fax ++ 1 513 336 7260
Toll Free 800 787 5426
eMail info@haag-streit-usa.com
internet www.haag-streit-usa.com